

# FloorBoost

**Inflatable Lifting Cushion**

## **Instruction Manual**

*WARRANTY, AND PRODUCT INFORMATION*

**CONGRATULATIONS**

**YOU GOT BOOSTED!**



The FloorBoost is an innovative inflatable lifting cushion created to offer vital assistance to the elderly enabling them to safely and effortlessly get up after a fall to the floor.

*It is highly recommended to set up the FloorBoost and familiarize yourself with its features before using it to lift someone off the floor.*

# GETTING TO KNOW YOUR FLOORBOOST



CHAIR SIDE VIEW



CHAIR BACK VIEW

## SETTING UP YOUR FLOORBOOST

### SETUP INSTRUCTIONS & USE:

1. Remove the FloorBoost from its packaging and unfold it.
2. Locate the air valve situated at the back of the chair.
3. Twist and remove the valve cap. *(See Air Valve Instructions below)*
4. Engage the air valve spring by pressing it down. This opens the valve.
5. Read the Instruction manual for the Air-Pump.
6. Attach the air-hose to the air valve. Attach the hose to the pump.
7. The individual on the floor should move towards the seating area designated for lifting. You'll notice a sign on the FloorBoost that reads "Sit Here." Place feet in the direction indicated on the sticker.
8. Activating the Electric Pump  
To start the electric pump, simply press and hold the power buttonⓈ for several seconds. Exercise caution to prevent over-inflation, targeting a maximum pressure of 8 PSI. The inflation process generally lasts around 1 to 3 minutes.
  - The person being lifted should push back with their feet to facilitate the front end rising.
  - Alternatively, a caregiver can assist by gently pulling back on the backrest.
9. Once the lift chair is fully inflated, the individual should move their legs to the **left** or **right**. Turning sideways in the chair allows you to utilize the backrest to assist in rising to a standing position.

# HOW TO OPERATE THE AIR VALVE

## AIR VALVE INSTRUCTIONS

1. Remove the valve cap.
2. Push the valve spring down. This allows the device to be inflated. (See diagram below)
3. Connecting the Hose for Inflation
  - Attach the hose to the air valve located at the rear of the chair.
  - Apply pressure to ensure the nozzle is properly aligned, then turn it clockwise to secure it in place.
  - Connect the other end of the hose to the inflation port on the air pump.

Once everything is fastened, initiate the inflation process by pressing and holding the power button. ⏻

4. Once you achieve the desired PSI level, a maximum of 8 PSI, simply turn off the pump. The air will remain trapped within the device.

## DEFLATION INSTRUCTIONS

1. Remove the air-hose from the pump. There will be a gush of air for several seconds. Place the air-hose on the deflation port on the air-pump. Turn the pump on to remove the remainder of the air.


2. Roll the FloorBoost up and secure the nylon strap. The FloorBoost is ready for transport.

### About the Air Pump

The air pump operates in **two stages**.

The **first stage** is relatively quiet and inflates up to 1 PSI.

Once the **second stage** is activated, the pump will generate a deep rumbling noise as it continues to inflate.

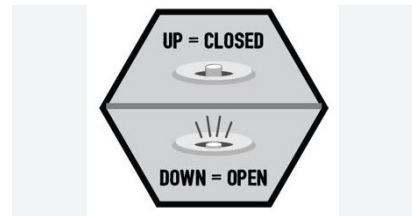

## WARNING & GUIDELINES!

Users must recognize and take all necessary precautions to avoid misuse and prevent accidents or damage. It is crucial to perform regular inspections to assess wear, identify damage, or determine if adjustments are needed.

The FloorBoost is strictly intended for its designed purposes only. Any misuse beyond these specifications is not allowed. Not adhering to the provided instructions or usage guidelines will result in the warranty being voided.

Never place an infant younger than 15 months old on the FloorBoost product, as it presents a suffocation hazard. Additionally, there is a risk of entrapment in vertical areas. Always ensure that children are supervised when they are in proximity to the product.

Steer clear of using flammable materials, such as aerosol tire repair products, to inflate your FloorBoost, as they can pose a risk of explosion or ignition.

Keep away from fire or extreme heat

DO NOT stand or jump on the FloorBoost product.

DO NOT use chemicals on your FloorBoost. Mild soap and water should be used when cleaning.

Never use this product as a personal flotation device or for lifesaving purposes. These products are not intended for use in water.

DO NOT exceed the recommended PSI, as doing so poses a risk of explosion, bodily injury, or even death. This danger must be taken seriously when utilizing electric or battery-powered inflation pumps.

## LIMITED WARRANTY!

FloorBoost offers a limited 5 year limited warranty on all products branded with the FloorBoost name, effective from the purchase date for the original buyer. This warranty is applicable solely to items sold by FloorBoost and covers only manufacturing defects. To start a warranty claim, the original purchaser must contact our support team via email (support@bedboost.com) to obtain a Return Authorization Number (RA#).

## RETURN POLICY

Changed your mind? No problem! FloorBoost gladly accepts returns for any reason within 30 days of receiving your order. You will receive a full refund of the purchase price, minus a 10% restocking fee.

Return Conditions:

- All returned items must be unused and in new condition.
- Include the original user manuals, accessories, and any other documentation or parts.
- If items are used or cannot be resold as new, FloorBoost will return them to the buyer at their expense. \*(Please note that you are responsible for shipping charges.)

To obtain a return authorization number (RA#), please contact the FloorBoost Customer Service team at support@bedboost.com. Make sure to include proof of purchase, such as the original invoice or receipt. The support team will then send you the RA#.

## EXCHANGES & REFUNDS

### Exchanges

To swiftly acquire a new item, please reach out to us at support@bedboost.com. We will provide you with a Return Authorization Number (RA#) so you can return your current product. Once we accept the return, we will dispatch the new item to you. Please be aware that a 10% restocking fee will apply to exchanges to cover materials and time required to repackage the item for resale. ( You are responsible for shipping charges back to FloorBoost)

### Refunds

We will notify you once we have received and inspected your return, and inform you whether your refund has been approved. If approved, the refund will be automatically processed to your original payment method. Please note that your bank or credit card company may take some time to process and post the refund.